

MIBS Stretcher Cleaning Instructions

<https://www.rescueandmedical.com/mibs-stretcher/>



Casualty Isolation Sheet, Insulated Vapour Sheet, Casualty Head Restraint, Carry Bag and Detachable Straps*

Step 1: Machine wash with detergent at 60°C.

Step 2: Hang up to dry.

*We recommend placing the detachable straps in a laundry mesh bag (not provided).

Stretcher, Spinal Insert, Skid Sheet and Carry Tubes

Step 1: Lay stretcher out flat to identify which cleaning processes are required.

Step 2:

- A. Dirt or mud:** Use deck brush or mop with a mixture of mild detergent mixed with warm water to scrub the stretcher.
- B. Oil or grease stains:** Use a mild degreasing fluid or spray (as used to degrease motor vehicle engines etc.) on the local area of contamination.
- C. Decontamination of body fluids:** Use a disinfectant cleaning solution spray such as:

Bioguard disinfectant cleaning solution spray

<https://www.bioguardhygiene.co.uk/products/disinfectant-cleaning-solution/>
https://www.bioguardhygiene.co.uk/wp-content/uploads/2018/06/00736-Bioguard-Website-Data-Sheets_Disinfectant-Cleaning-Solution.pdf



Rely+On® Virkon® disinfectant 1% solution spray

<https://lanxess.com/en/products-and-brands/brands/rely-on-disinfection/rely-on-products/rely-on-virkon>
<https://www.day-impex.co.uk/laboratory/virkon-disinfectant/> inc. Safety Data Sheets for solution, tablets & powder forms
https://youtu.be/n4FX9Zr_CDU?si=inr5lfU2qH3oCVm Video on how to prepare and use the spray solution
https://youtu.be/ort5GQV0mU4?si=3i_M5gqK90i2JvkA Video on creating the solution from tablets
<https://www.day-impex.co.uk/wp-content/uploads/RelyOn-Virkon-Laboratory.pdf>
<https://my.supplychain.nhs.uk/catalogue/product/kwd1396>



When using cleaning products always follow the manufacturer's user instructions concerning personal protection, preparation, dilutions and application. Apply solution on surface with a trigger spray bottle. Wait for a minimum of 10 minutes. Scrub the area with a brush or sponge.

Step 3: Wash off or power wash with water (no steam).

Step 4: Hang up to dry.